

INSTRUCTIONS FOR USE

Ref: APEX-IFU-AUB1000

Description	Adaptor Urinal Bulbous Disposable Medical Pulp
Product Part Number	AUB1000
Ordering Item Code	100xxxAUB1000 (xxx – To be assigned upon order)
UDI-DI	95510204301752

Product Image



Intended Use

For patient treatment and monitoring, the Adaptor for Urinal Bulbous is made from eco-friendly biodegradable kraft material, intended for use as an adaptor for converting male urinal bulbous to female urinal bulbous. The contoured opening is designed to fit comfortably when placed between the legs of female patients in either sitting or laying positions.


Storage

Keep in a cool, dry place, away from heat, flames, and sunlight.

User Guide




Insert Adaptor into Male Urinal Bulbous converting it as Female Urinal Bulbous

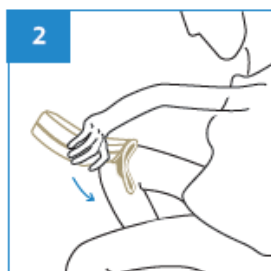

Place the Female Urinal Bulbous between the legs and cover completely for urination.



Dispose the Female Urinal Bulbous directly into macerator or empty the content from the Female Urinal Bulbous before disposing into appropriate waste bins.

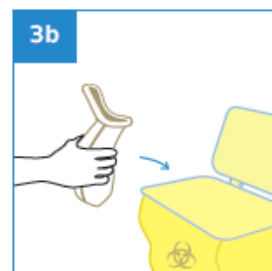

WARNING

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.



APEX PULP Pte Ltd

Company Registration No.: 200715327H
132 Joo Seng Road #04-01. Singapore 368358
Tel: +65 6817 8969 | Fax: +65 6636 9813
www.apexpulp.com | sales@apexpulp.com

