

INSTRUCTIONS FOR USE

Ref: APEX-IFU-BPS2000

Date: 20/08/2022

Revision Date: 30/09/2025

Revision No.: 1

Description: Bedpan Support < 100kg Disposable Medical Pulp
Product Part Number: BPS2000
Ordering Item Code: 100APPBPS2000


Product Image




Intended Use

The Bedpan Support is designed for patient treatment and monitoring, made from eco-friendly and biodegradable kraft paper materials. It serves as a sturdy support for a bedpan liner, capable of withstanding a patient weight of up to 50 kg. For additional strength, multiple supports can be stacked together.


User Guide

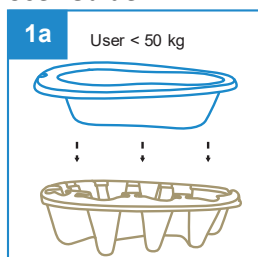


For user of under 50 kg, Place bedpan liner over the disposable moulded pulp bedpan support.

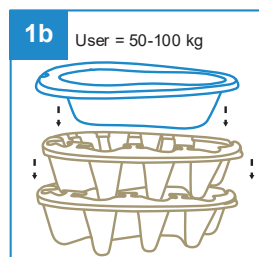

If the user weight is more than 50 kg, another piece can be stacked for added reinforcement.

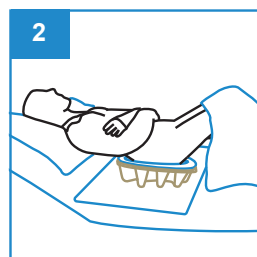

Slide the bedpan liner with support under the buttock. To prevent spills, place a sheet of incontinence pad under the bedpan.

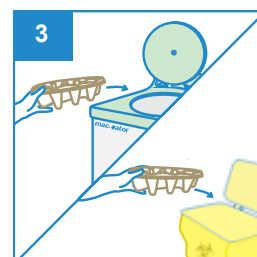

Dispose the Bedpan Liner with disposable support directly into macerator or empty the content from bedpan liner before disposing into appropriate waste bins.



WARNING

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.

