

## INSTRUCTIONS FOR USE

Ref: APEX-IFU-GMJ1000

Description	Graduated Measuring Jug 1000ml Disposable Medical Pulp
Product Part Number	GMJ1000
Ordering Item Code	100xxxGMJ1000 (xxx – To be assigned upon order)
UDI-DI	95510204301066


### Product Image




Intended Use	For patient treatment and monitoring, the Graduated Measuring Jug with 1L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as measuring jug to determine the amount of urine or fluid as it is marked with graduation levels ranging from 50ml to 1000ml.
--------------	--

Storage	Keep in a cool, dry place, away from heat, flames, and sunlight.
---------	--

### User Guide

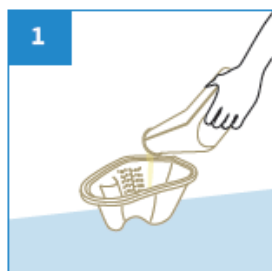


Place the Graduated Measuring Jug on a flat surface, pour liquid into it slowly to prevent splashes.

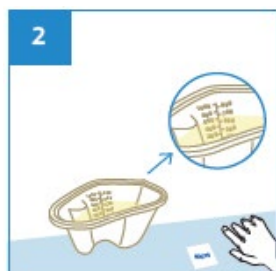

Read and record volume of the liquid from the level markings.



Dispose the Graduated Measuring Jug directly into macerator or empty liquid from the jug before disposing into appropriate waste bins.

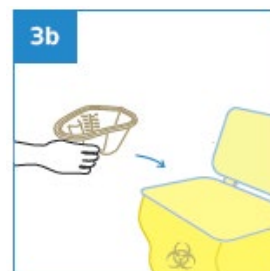

### WARNING

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

### Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.



### APEX PULP Pte Ltd

Company Registration No.: 200715327H  
132 Joo Seng Road #04-01. Singapore 368358  
Tel: +65 6817 8969 | Fax: +65 6636 9813  
www.apexpulp.com | sales@apexpulp.com

