

## INSTRUCTIONS FOR USE

Ref: APEX-IFU-GSB1500

Description	General Semi Bowl 1500ml Disposable Medical Pulp
Product Part Number	GSB1500
Ordering Item Code	100xxxGSB1500 (xxx – To be assigned upon order)
UDI-DI	95510204300762


### Product Image




Intended Use	For patient treatment and monitoring, the General Semi Bowl Graduated with 1.5L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as vomit bowl or sampling of urine and/or faeces by placing it over toilet bowl.
--------------	--

Storage	Keep in a cool, dry place, away from heat, flames, and sunlight.
---------	--


### User Guide

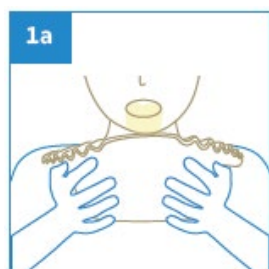


For vomiting, hold the General Semi Bowl with both hands below the chin and vomit into it.

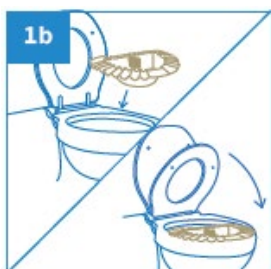

For sampling, raise the toilet seat lid, place the General Semi Bowl over the toilet bowl (place front for urint or rear for faeces) and lower the lid.

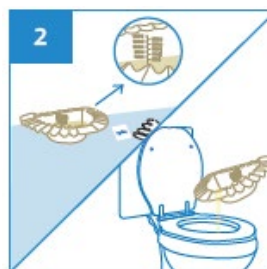

Read and record volume of the liquid from the level markings.

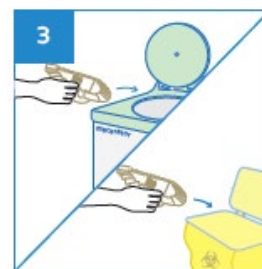

Dispose the General Semi Bowl directly into macerator or empty liquid from the bowl before disposing into appropriate waste bins.



### WARNING

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

### Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.



**APEX PULP Pte Ltd**

Company Registration No.: 200715327H  
132 Joo Seng Road #04-01. Singapore 368358  
Tel: +65 6817 8969 | Fax: +65 6636 9813  
www.apexpulp.com | sales@apexpulp.com

