

INSTRUCTIONS FOR USE

Ref: APEX-IFU-GWB3000

Description	General Wash Bowl 3000ml Disposable Medical Pulp
Product Part Number	GWB3000
Ordering Item Code	100xxxGWB3000 (xxx – To be assigned upon order)
UDI-DI	95510204302810

Product Image



Intended Use

For patient treatment and monitoring, the General Purpose Bowl Graduated with 3L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as a general bowl to rinse towels and/or to hold several litres of water. It is marked with graduation levels ranging from 500ml to 3000ml. Do **NOT** dispose towel into macerator.

Storage

Keep in a cool, dry place, away from heat, flames, and sunlight.


User Guide




1
Dispense and apply soap directly on the patient's body or on a towel.



2
Rinse the towel in the General Wash Bowl to wipe and cleanse patient's body.

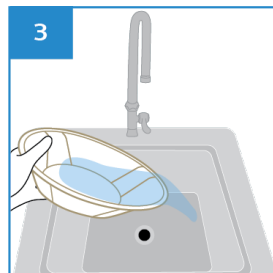

3
After use, empty the contents into the sink.

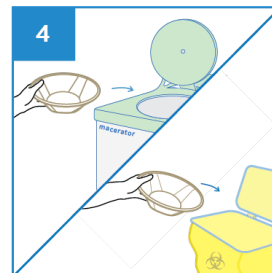

4
Dispose the General Purpose Bowl directly into macerator or into appropriate waste bins.



WARNING

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.



APEX PULP Pte Ltd

Company Registration No.: 200715327H
132 Joo Seng Road #04-01. Singapore 368358
Tel: +65 6817 8969 | Fax: +65 6636 9813
www.apexpulp.com | sales@apexpulp.com

