

INSTRUCTIONS FOR USE

Ref: APEX-IFU-LBP2000

Description	Lid for Bedpan Liner Disposable Medical Pulp
Product Part Number	LBP2000
Ordering Item Code	100xxxLBP2000 (xxx – To be assigned upon order)
UDI-DI	95510204302988

Product Image




Intended Use

Designed to cover our Bedpan Liner Medical Pulp (BPG2000) to prevent splashes and hide waste contents from sight and smell. Made from biodegradable paper material, it is macerator-friendly for safe and sustainable disposal. This lid can also be used with the Midi Slipper Pan Liner (MSP1300). Do **NOT** dispose of any plastic supports into the macerator.


Storage

Keep in a cool, dry place, away from heat, flames, and sunlight.

User Guide

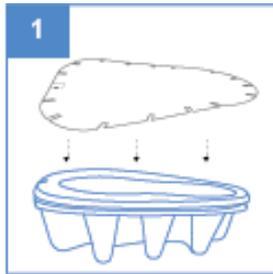

After use, remove the Bedpan Liner with support, place a lid cover over it to contain the odour and sight.

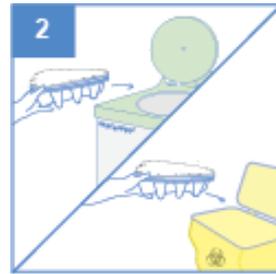

Dispose the Bedpan Liner with disposable support directly into macerator or empty the content from Bedpan Liner before disposing into an appropriate waste bin.



WARNING

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.



APEX PULP Pte Ltd

Company Registration No.: 200715327H
132 Joo Seng Road #04-01. Singapore 368358
Tel: +65 6817 8969 | Fax: +65 6636 9813
www.apexpulp.com | sales@apexpulp.com

