

## INSTRUCTIONS FOR USE

Ref: APEX-IFU-LGP2000

Description	Lid for General Purpose Pot Disposable Medical Pulp
Product Part Number	LGP2000
Ordering Item Code	100xxxLGP2000 (xxx – To be assigned upon order)
UDI-DI	95510204303046

### Product Image




### Intended Use

Designed to fit over the General Purpose Pot (GPP2000) to provide hygienic coverage. It features a central hole that allows caregivers to drain urine bags directly into the pot without splashing. Made from biodegradable paper material, it is macerator-friendly for eco-conscious disposal.


### Storage

Keep in a cool, dry place, away from heat, flames, and sunlight.


### User Guide

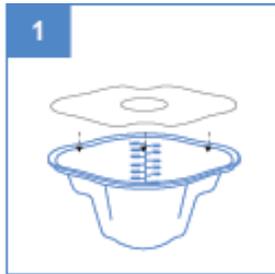

Place lid over General Purpose Pot.

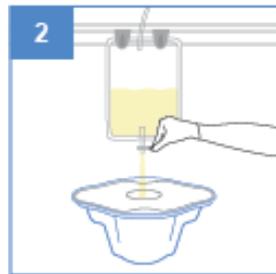

Place General Purpose Pot directly under urinal bag. Drain urine through the lid hole to prevent splashes.

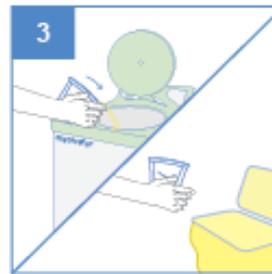

Dispose the General Purpose Pot with lid directly into macerator or into an appropriate waste bin.



### WARNING

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

### Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.



**APEX PULP Pte Ltd**

Company Registration No.: 200715327H  
132 Joo Seng Road #04-01. Singapore 368358  
Tel: +65 6817 8969 | Fax: +65 6636 9813  
www.apexpulp.com | sales@apexpulp.com

