

INSTRUCTIONS FOR USE

Ref: APEX-IFU-MSP2000

Description	Maxi Slipper Pan Liner 2000ml Disposable Medical Pulp
Product Part Number	MSP2000
Ordering Item Code	100xxxMSP2000 (xxx – To be assigned upon order)
UDI-DI	95510204300458

Product Image




Intended Use

For patient treatment and monitoring, the Maxi Slipper Pan Liner Graduated with 2L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as fracture receptacle for urine and/or faeces. To use the maxi slipper pan liner on the bed, place it over a plastic support. Do **NOT** dispose the plastic support into macerator.


Storage

Keep in a cool, dry place, away from heat, flames, and sunlight.


User Guide

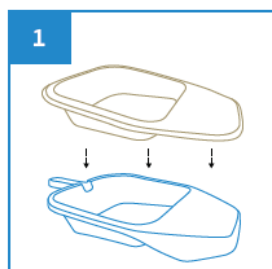


Place the Maxi Slipper Pan Liner over the plastic maxi slipper pan support.

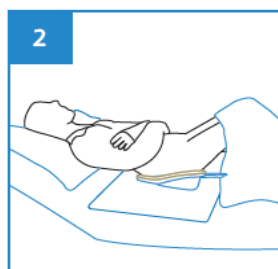

Slide the Maxi Slipper Pan Liner with support under the buttock. To prevent spills, place a sheet of incontinence pad under the Maxi Slipper Pan.

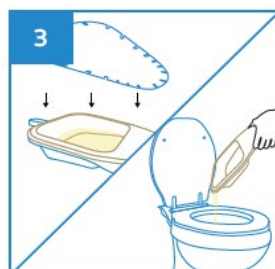

After use, remove the Maxi Slipper Pan Liner with support, place a lid cover over it to contain the odour and sight.

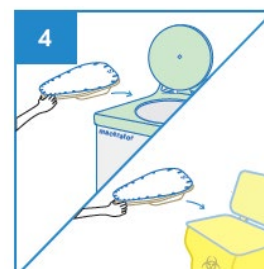

Dispose Maxi Slipper Pan Liner directly into macerator or empty the content from maxi slipper pan liner before disposing into appropriate waste bins.



WARNING

Any serious incident related to the device must be reported to the manufacturer and to the competent authority of the Member State in which the user is established.

Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.



APEX PULP Pte Ltd

Company Registration No.: 200715327H
132 Joo Seng Road #04-01. Singapore 368358
Tel: +65 6817 8969 | Fax: +65 6636 9813
www.apexpulp.com | sales@apexpulp.com

