

MAXI SLIPPER PAN LINER 2000ML

Eco-Friendly Medical Pulp



Made from
BIODEGRADABLE
Double Lined Kraft

**Designed for Single Use to
Reduce Risk of Infection**



Maxi Slipper Pan Liner Graduated 2000ml Biodegradable Medical Pulp

The Maxi Slipper Pan Liner with 2L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as fracture receptacle for urine and/or faeces. When using the maxi slipper pan liner on the bed, place it over a plastic support.

Manufactured in accordance with PAS29:1999 Disposable Pulp Products For Use in Healthcare.

Suitable for



Hospitals



Nursing Homes



Personal Use




Medical Clinics




Disability Homes


User Guide

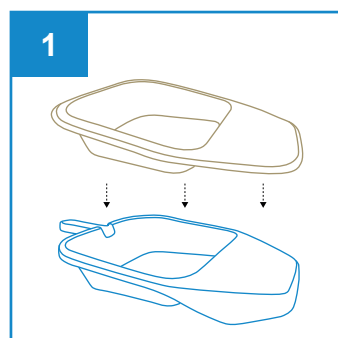


Place the Maxi Slipper Pan Liner over the plastic maxi slipper pan support.

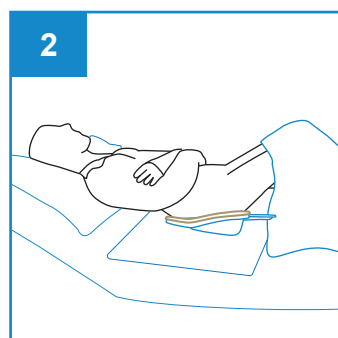

Slide the Maxi Slipper Pan Liner with support under the buttock. Place a sheet of incontinence pad under the Maxi Slipper Pan to prevent any spills.

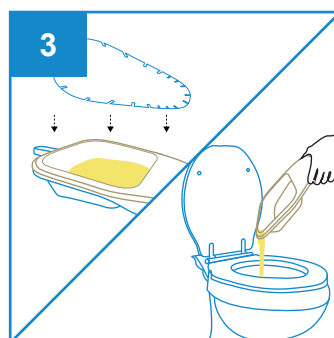

After use, remove the Maxi Slipper Pan Liner with support, place a lid cover over it to contain the odour and sight.

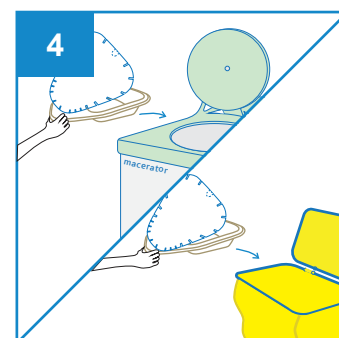

Dispose the Maxi Slipper Pan Liner directly into macerator or empty the content from the maxi slipper pan liner before disposing into appropriate waste bins.

Note: DO NOT dispose the plastic support into macerator.



Single-Use



Macerator
Friendly



Eco-Friendly



Water
Retention



Stackable



Recycled



No Direct
Sunlight



Keep Dry

PRODUCT DESCRIPTION

Maxi Slipper Pan Liner 2000ml

PRODUCT SIZE

370 x 270 x 85mm

PACKAGING

100 pcs per carton

CARTON DIMENSION

590 x 395 x 560mm

UDI-DI: 95510204300458

ORDER CODE: 100xxxMSP2000 (xxx to be assigned upon ordering)



APEX PULP PTE LTD

Company Registration No.: 200715327H
132 Joo Seng Road #04-01, Singapore 368358
Tel: +65 6817 8969 | Fax: +65 6636 9813
www.apexpulp.com | sales@apexpulp.com

DOC. REF: APEX-PIB-MSP2000

Date of Revision: 01/09/2026

Revision: 6

© 2026 APEX PULP PTE LTD. All rights reserved.