

ADAPTOR FOR URINAL BULBOUS


Eco-Friendly Medical Pulp



Designed for Single Use to Reduce Risk of Infection



Adaptor For Urinal Bulbous Eco-Friendly Medical Pulp

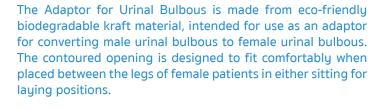

Manufactured as single-use disposable medical pulp to meet the requirement to hold water and/or human waste for a minimum period of four hours at an initial temperature of 35°C. Suitable for disposal into macerator which complies with PAS29:1999 "Disposable Pulp For Use in Healthcare".





Urinal

94/boy

Suitable for



Hospitals



Nursing Homes



Personal Use



Disability Homes

3b


User Guide



Insert Adaptor into Male Urinal Bulbous converting it as Female **Urinal Bulbous**



Place the Female Urinal Bulbous between the legs and cover completely for urination.

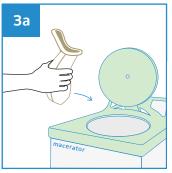

Dispose the Female Urinal Bulbous directly into macerator or empty the content from the Female Urinal Bulbous before disposing into appropriate waste bins.













PRODUCT DESCRIPTION

PRODUCT	SIZE

PACKAGING

CARTON DIMENSION

ORDER CODE

Adaptor for Urinal Bulbous (Female Urinal)

120 x 63 x 125 mm

200 pcs per carton

510 x 260 x 210 mm

100APPAUB1000









Manufactured in accordance with PAS29:1999 Disposable Pulp Products For Use in Healthcare