



**APEX PULP**

Manufacturer of Moulded Pulps

# **BEDPAN LINER GRADUATED 2000ML Eco-Friendly Medical Pulp**



Made from  
**BIODEGRADABLE**  
Double Lined Kraft

Designed for Single Use to  
Reduce Risk of Infection



The Bedpan Liner Graduated with 2L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as receptacle for urine and/or faeces. To use the bedpan liner, place it over a plastic or moulded pulp support. Alternatively, the bedpan liner can also be placed under commode chair for toileting.

Manufactured as single-use disposable medical pulp to meet the requirement to hold water and/or human waste for a minimum period of four hours at an initial temperature of 35°C. Suitable for disposal via macerator which complies with PAS29:1999 "Disposable Pulp For Use in Healthcare".

**Suitable for**



Hospitals



Nursing Homes



Personal Use




Medical Clinics




Disability Homes


**User Guide**

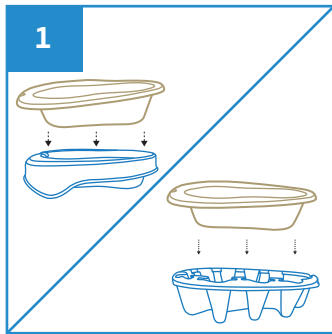


Place Bedpan Liner over the plastic or disposable moulded pulp bedpan support.

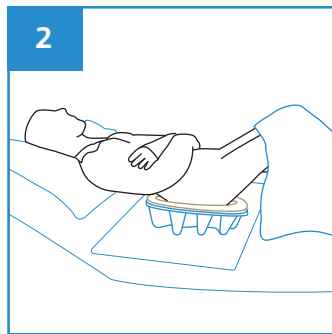

Slide the Bedpan Liner with support under the buttock. To prevent spills, place a sheet of incontinence pad under the bedpan.

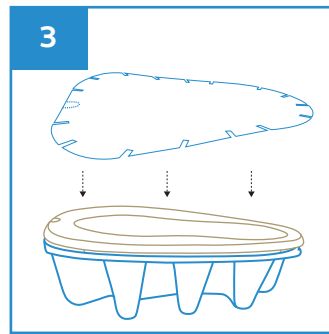

After use, remove the Bedpan Liner with support, place a lid cover over it to contain the odour and sight.

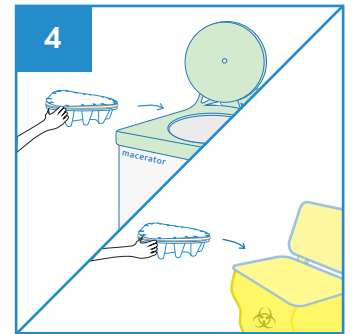

Dispose the Bedpan Liner with disposable support directly into macerator or empty the content from Bedpan Liner before disposing into appropriate waste bins.

Note: DO NOT dispose the plastic support into macerator.



Stackable



Water Retention



Macerator Friendly



Eco-Friendly



Recyclable

PRODUCT DESCRIPTION	PRODUCT SIZE	PACKAGING	CARTON DIMENSION	ORDER CODE
Bedpan Liner Graduated 2000ml	360 x 300 x 100 mm	100 pcs per carton	755 x 370 x 310 mm	100APPBPG2000



Manufactured in accordance with PAS29:1999  
Disposable Pulp Products For Use in Healthcare

Rev1(1123)EN

