

BEDPAN LINER

GRADUATED 2000 ML

Eco-Friendly Medical Pulp



Made from
BIODEGRADABLE
Double Lined Kraft

**Designed for Single Use to
Reduce Risk of Infection**

Bedpan Liner Graduated 2000ml Biodegradable Medical Pulp

The Bedpan Liner Graduated with 2L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as receptacle for urine and/or faeces. To use the bedpan liner, place it over a plastic or moulded pulp support. Alternatively, the bedpan liner can also be placed under com-mode chair for toileting.

Manufactured in accordance with PAS29:1999 Disposable Pulp Products For Use in Healthcare.

Suitable for



Hospitals



Nursing Homes



Personal Use




Medical Clinics




Disability Homes


User Guide

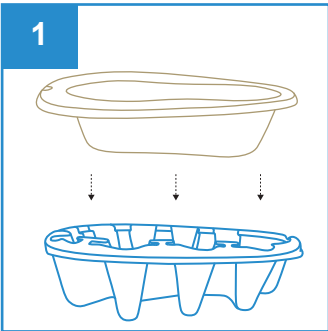


Place Bedpan Liner over the plastic or disposable moulded pulp bedpan support.

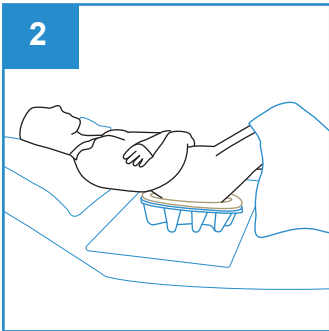

Slide the Bedpan Liner with support under the buttock. To prevent spills, place a sheet of incontinence pad under the bedpan.

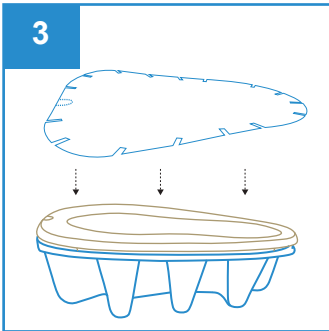

After use, remove the Bedpan Liner with support, place a lid cover over it to contain the odour and sight.

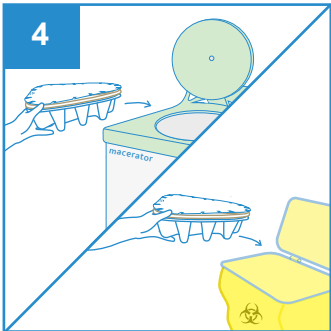

Dispose the Bedpan Liner with disposable support directly into macerator or empty the content from Bedpan Liner before disposing into appropriate waste bins.

Note: DO NOT dispose the plastic support into macerator.



Single-Use



Macerator
Friendly



Eco-Friendly



Water
Retention



Stackable



Recycled



No Direct
Sunlight



Keep Dry

PRODUCT DESCRIPTION	PRODUCT SIZE	PACKAGING	CARTON DIMENSION
Bedpan Liner Graduated 2000ml	360 x 300 x 100mm	100 pcs per carton	755 x 370 x 310mm

UDI-DI: 95510204300076

ORDER CODE: 100xxxBPG2000 (xxx to be assigned upon ordering)



APEX PULP PTE LTD
Company Registration No.: 200715327H
132 Joo Seng Road #04-01, Singapore 368358
Tel: +65 6817 8969 | Fax: +65 6636 9813
www.apexpulp.com | sales@apexpulp.com

DOC. REF: APEX-PIB-BPG2000
Date of Revision: 09/01/2026
Revision: 3

© 2026 APEX PULP PTE LTD. All rights reserved.