

MIDI SLIPPER PAN LINER GRADUATED 1300ML

Eco-Friendly Medical Pulp



Designed for Single Use to Reduce Risk of Infection



Midi Slipper Pan Liner Graduated 1300ml **Eco-Friendly Medical Pulp**





The Midi Slipper Pan Liner with 1.3L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as fracture receptacle for urine and/or faeces. To use the midi slipper pan liner on the bed, place it over a plastic support.

Manufactured as single-use disposable medical pulp to meet the requirement to hold water and/or human waste for a minimum period of four hours at an initial temperature of 35°C. Suitable for disposal via macerator which complies with PAS29:1999 "Disposable Pulp For Use in Healthcare".

Suitable for






Nursing Homes









User Guide

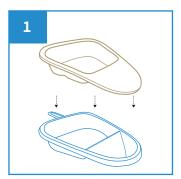


Place the Midi Slipper Pan Liner over the plastic midi slipper pan support.

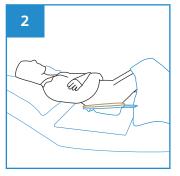

Slide the Midi Slipper Pan Liner with support under the buttock. To prevent spills, place a sheet of incontinence pad under the Midi Slipper Pan.

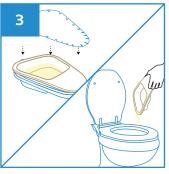

After use, remove the Midi Slipper Pan Liner with support, place a lid cover over it to contain the odour and sight.

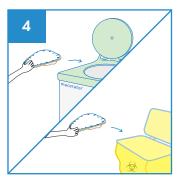

Dispose the Midi Slipper Pan Liner directly into macerator or empty the content from the midi slipper pan liner before disposing into appropriate waste bins.

Note: DO NOT dispose the plastic support into macerator.











PRODUCT DESCRIPTION	PRODUCT SIZE	PACKAGING	CARTON DIMENSION	ORDER CODE
Midi Slipper Pan Liner Graduated 1300ml	340 x 290 x 85 mm	100 pcs per carton	580 x 360 x 420 mm	100APPMSP1300









Manufactured in accordance with PAS29:1999 Disposable Pulp Products For Use in Healthcare