

MIDI SLIPPER PAN LINER GRADUATED 1300ML Eco-Friendly Medical Pulp



Made from
BIODEGRADABLE
Double Lined Kraft

Designed for Single Use to
Reduce Risk of Infection

Midi Slipper Pan Liner Graduated

1300ml Biodegradable Medical Pulp

The Midi Slipper Pan Liner with 1.3L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as fracture receptacle for urine and/or faeces. To use the midi slipper pan liner on the bed, place it over a plastic support.

Manufactured in accordance with PAS29:1999 Disposable Pulp Products For Use in Healthcare.

Suitable for



Hospitals



Nursing Homes



Personal Use




Medical Clinics




Disability Homes


User Guide

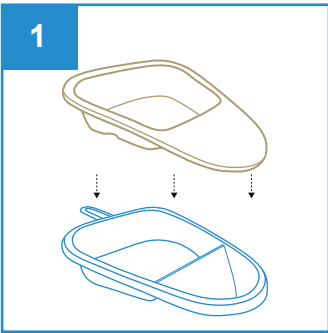


Place the Midi Slipper Pan Liner over the plastic midi slipper pan support.

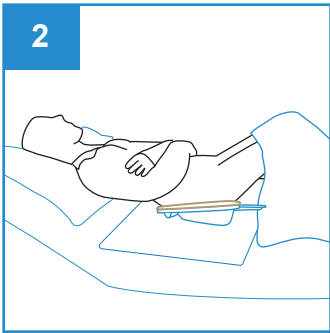

Slide the Midi Slipper Pan Liner with support under the buttock. To prevent spills, place a sheet of incontinence pad under the Midi Slipper Pan.

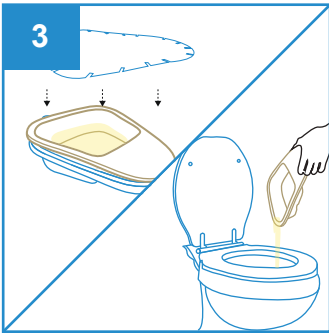

After use, remove the Midi Slipper Pan Liner with support, place a lid cover over it to contain the odour and sight.

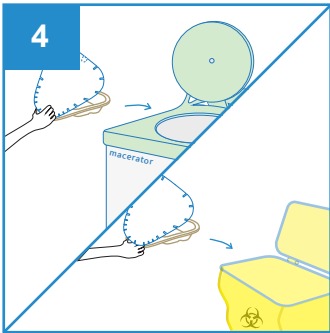

Dispose the Midi Slipper Pan Liner directly into macerator or empty the content from the midi slipper pan liner before disposing into appropriate waste bins.

Note: DO NOT dispose the plastic support into macerator.



Single-Use



Macerator
Friendly



Eco-Friendly



Water
Retention



Stackable



Recycled



No Direct
Sunlight



Keep Dry

PRODUCT DESCRIPTION	PRODUCT SIZE	PACKAGING	CARTON DIMENSION
Midi Slipper Pan Liner 1300ml	345 x 305 x 80mm	100 pcs per carton	580 x 360 x 420mm

UDI-DI: 95510204300212

ORDER CODE: 100xxxMSP1300 (xxx to be assigned upon ordering)

