

INSTRUCTIONS FOR USE

Ref: APEX-IFU-BPS2000

Description Bedpan Support < 100kg Disposable Medical Pulp

Product Part Number BPS2000

Ordering Item Code 100APPBPS2000


Product Image




Intended Use

For the purpose of patient treatment and monitoring, the sturdy Bedpan Support that supports up to 100 kg evenly distributed weight, is made from eco-friendly biodegradable kraft material, intended for use as support with bedpan liner place over it. Due to the innovative mould pulp design structure, it can withstand patient weighing up to 100 kg when placed under the buttock.


User Guide

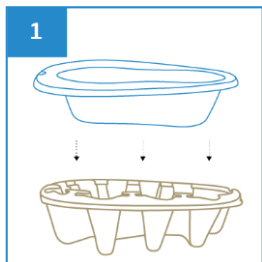


1 Place bedpan liner over the disposable moulded pulp bedpan support.

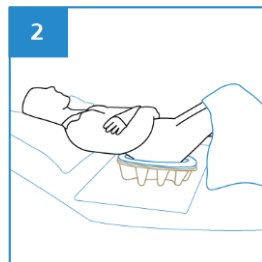

2 Slide the bedpan liner with support under the buttock. To prevent spills, place a sheet of incontinence pad under the bedpan.

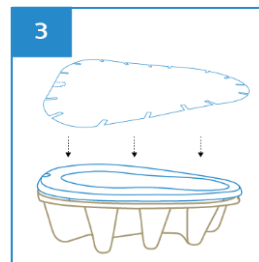

3 After use, remove the bedpan liner with support, place a lid cover over it to contain the odour and sight.

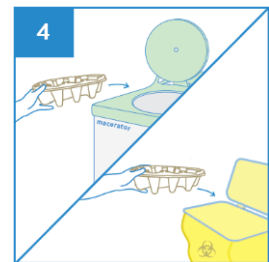

4 Dispose the Bedpan Liner with disposable support directly into macerator or empty the content from bedpan liner before disposing into appropriate waste bins.

Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.

Rev2(1123)

