

INSTRUCTIONS FOR USE

Ref: APEX-IFU-GPB3000

Description General Purpose Bowl 3000ml Disposable Medical Pulp

Product Part Number GPB3000

Ordering Item Code 100APPGPB3000


Product Image



Intended Use For the purpose of patient treatment and monitoring, the General Purpose Bowl Graduated with 3L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as a general bowl to rinse towels and/or to hold several litres of water. It is marked with graduation levels ranging from 500ml to 3000ml.

Caution Do not dispose the towel into macerator

User Guide

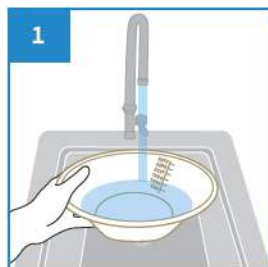


1 Fill the General Purpose Bowl with water to the required level.



2 Wet or rinse the towel with water inside the bowl.



3 After use, pour away the water.

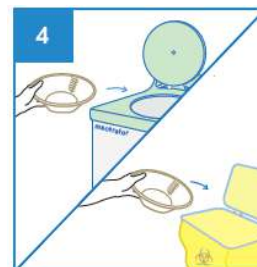

4 Dispose the General Purpose Bowl directly into macerator or into appropriate waste bins.

Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.

Rev2(1123)

