

INSTRUCTIONS FOR USE

Ref: APEX-IFU-GPP2000

Description General Purpose Pot 2000ml Disposable Medical Pulp

Product Part Number GPP2000

Ordering Item Code 100APPGPP2000


Product Image




Intended Use

For the purpose of patient treatment and monitoring, the General Purpose Pot Graduated with 2L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as container to drain urine and/or to determine the amount of urine or fluid as it is marked with graduation levels ranging from 50ml to 2000ml.

User Guide

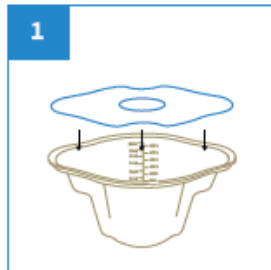


1 Place lid over General Purpose Pot.

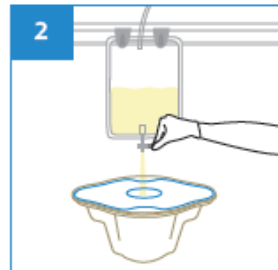

2 Place General Purpose Pot under urinal bag. Drain urine through the lid hole to prevent splashes.



3 To measure contents, remove the lid and read the volume from graduated markings.

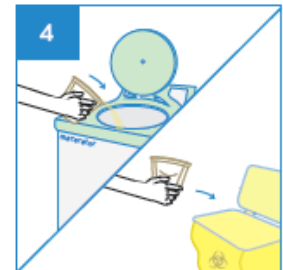

4 Dispose the General Purpose Pot with lid directly into macerator or into appropriate waste bins.

Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.

Rev2(1123)

