

INSTRUCTIONS FOR USE

Ref: APEX-IFU-MSP2000

Description Maxi Slipper Pan Liner 2000ml Disposable Medical Pulp

Product Part Number MSP2000

Ordering Item Code 100APPMSP2000


Product Image




Intended Use For the purpose of patient treatment and monitoring, the Maxi Slipper Pan Liner Graduated with 2L rated capacity is made from eco-friendly biodegradable kraft material, intended for use as fracture receptacle for urine and/or faeces. To use the maxi slipper pan liner on the bed, place it over a plastic support.

Caution Do not dispose the plastic support into macerator


User Guide

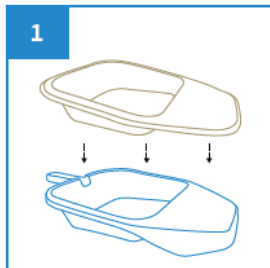


1 Place the Maxi Slipper Pan Liner over the plastic maxi slipper pan support.

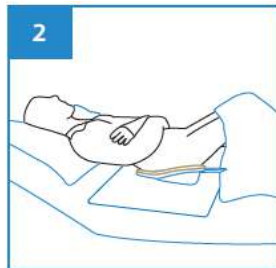

2 Slide the Maxi Slipper Pan Liner with support under the buttock. To prevent spills, place a sheet of incontinence pad under the Maxi Slipper Pan.

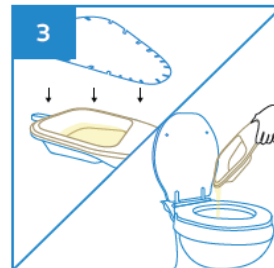

3 After use, remove the Maxi Slipper Pan Liner with support, place a lid cover over it to contain the odour and sight.

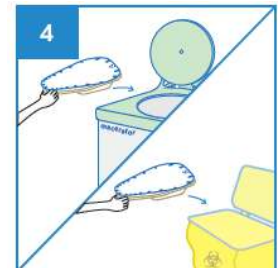

4 Dispose Maxi Slipper Pan Liner directly into macerator or empty the content from maxi slipper pan liner before disposing into appropriate waste bins.

Disclaimer:

All products specifications, illustrations, drawings, images, dimensions, performance data and other information on our Instructions for Use or made available by us are subjected to change without notice and they are intended to represent no more than a general illustration of the products intended use and their features and do not constitute a warranty or representation by us that the products will conform with the same. Products that are shipped to you may vary from these images or specifications.

